



Watch Video

# HALO®

## Folding The HALO® For Storage

1. Fold the locking strap approximately in half.



2. Rotate the folded locking strap and lay it flat against the top of the lever.



3. Rotate lever flat against the base as shown.



4. Holding the HALO base in one hand, use the other hand to pull strap into a triangle as shown.



5. Place the point of the triangle at the center of the base bottom.




6. Fold the side without the finger loop across the bottom of base.



7. Fold the side with the finger loop across the base and press flat.



8. Store HALO in a dry place. Avoid long term exposure to direct sunlight or water.

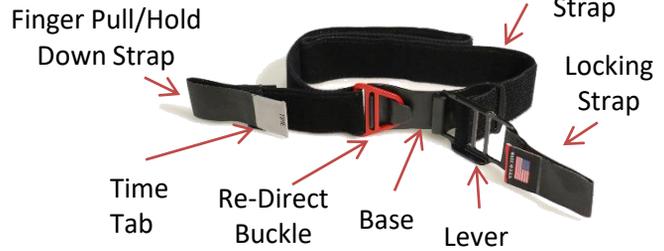

HALO®

### **CAUTION:**

**THIS IS A SINGLE USE PRODUCT.**  
**THIS IS NOT A CONSUMER PRODUCT.**

Read this manual before using this product. Failure to read, understand and follow the instructions and safety precautions in this manual can result in serious injury or death. Keep this manual in a safe location for future reference. The use of any tourniquet is as a last resort and should only be employed when bleeding cannot be controlled, and the loss of blood is life threatening. This product may only be used in strict adherence to these instructions and applied only as directed by EMS personnel or under the supervision of a medical professional. Inappropriate application of this product may result in serious injury or loss of limb. This product is NOT intended for sale to the general consumer public.

HALO®

### **Limitation of Liability**

Tactical, L.L.C., its employees, agents, contractors, suppliers, and distributors shall assume NO liability for injury or damages arising from the application and use of the tourniquet.

The user assumes all risk of liability.

# HALO®

Hemorrhage Arresting Lever Operated  
**TOURNIQUET**



[WWW.HALOTQ.com](http://WWW.HALOTQ.com)

EC REP



Poly-Tech Industrial Inc.  
13728 Statesville Rd.  
Huntersville, NC 28078 USA

Proudly Manufactured in the U.S.A.

Patented



All Rights Reserved